



ENMED® ENM - 603 Examination and Protective Gloves

The ENMED ENM - 603 Protective Gloves which are used to protect the user against dangerous chemicals and micro-organisms. The protective gloves are designed and manufactured so that under normal conditions of use without exposure to additional risks, except in the event of the user's individual hypersensitivity, the wearers can perform the activity as normally as possible with appropriate protection.

Applications

- Hospitals • Dormitories
- Nursing Homes
- Hotels / Guest Houses
- Food manufacturing

Storage and Disposal

- Store in dry, clean conditions in original packaging, away from direct sunlight, sources of high temperature, and solvent vapours
- Store within the temperature range -5°C to +45°C and with relative humidity below min %40 max80% Rh.
- Expected shelf life is five years from date of manufacture when stored as stated
- Replace garments if damaged, heavily contaminated or in accordance with local work practice or regulations
- Handle and dispose of contaminated garments with care and in accordance with applicable regulations.
- CE approved under PPE Directive (EU) 2016/425, Category III, MDD 93/42/EEC Class I.

Comfort and Protection



EN 420:2003
+A1:2009



EN ISO 374-1:2016/
A1:2018



EN ISO
374-5:2016



**EN
455**

EN 455



**AQL
1.5**



Limited Use



DO NOT
USE



CONSULT
INSTRUCTION
S
FOR USE



CAUTION

Size	Palm width
S	80 ± 10 mm
M	95 ± 10 mm
L	110 ± 10 mm
XL	≥ 110 mm

Materials: Nitrile Rubber

Thickness : 0.09 ± 0.02 mm

Color : Blue, Violet Blue, Black, Purple, White

- Latex Free
- Powder Free

Warnings and Limitations

Before use read and understand all user instructions and be sure that the product is suitable for the application and fitted correctly. Product must never be altered or modified. Prior to use inspect the clothing for any damage that could affect its protective function. Replace any damaged clothing. Care should be taken when removing contaminated garments, so as not to contaminate the user with any hazardous substances. If garments are contaminated, then decontamination procedures should be followed prior to removal of the garment. The wearing of chemical protective clothing may cause head stress if appropriate consideration is not given to the workplace environment. Appropriate under garments should be considered to minimize head stress or damage your garment.

Do not use for:

Contact with heavy oils, sparks or flame, or combustible liquids; Environments with high mechanical risks (abrasions, tears, cuts); Environments with conditions of excessive heat.

Central Adress

Muhsin Yaziciođlu Cad. Regnum
Sky Tower Kat.21 No.81
Cukurambar ANKARA- TURKIYE

Glove Factory

Saray Mahallesi Gıdacılar Cad. 3670 Ada 2 Parsel Kahraman Kazan /
ANKARA - TURKIYE

PACKAGE DESIGN



Size of Package

Package	225 x 115 x 65 mm
Box	235 x 240 x 335 mm

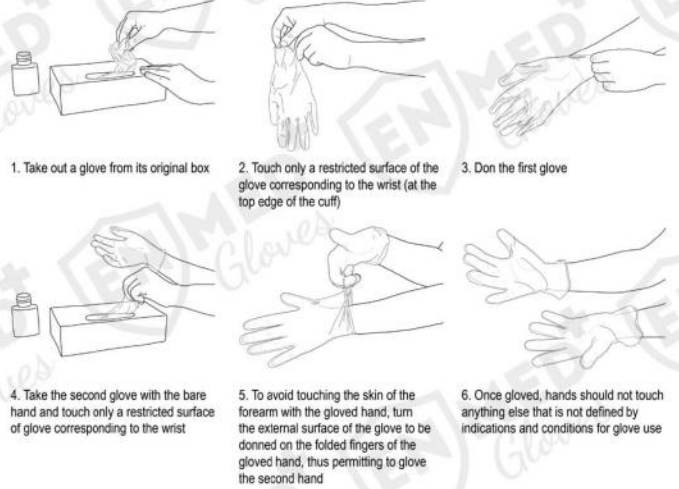
Packaging:

- 100 pieces per Box
- 10 Box per carton

USER GUIDE

When the hand hygiene indication occurs before a contact requiring glove use, perform hand hygiene by rubbing with an alcohol-based handrub or by washing with soap and water.

I. HOW TO DON GLOVES:

- 
1. Take out a glove from its original box
 2. Touch only a restricted surface of the glove corresponding to the wrist (at the top edge of the cuff)
 3. Don the first glove
 4. Take the second glove with the bare hand and touch only a restricted surface of glove corresponding to the wrist
 5. To avoid touching the skin of the forearm with the gloved hand, turn the external surface of the glove to be donned on the folded fingers of the gloved hand, thus permitting to glove the second hand
 6. Once gloved, hands should not touch anything else that is not defined by indications and conditions for glove use

II. HOW TO REMOVE GLOVES:

- 
1. Pinch one glove at the wrist level to remove it, without touching the skin of the forearm, and peel away from the hand, thus allowing the glove to turn inside out
 2. Hold the removed glove in the gloved hand and slide the fingers of the ungloved hand inside between the glove and the wrist. Remove the second glove by rolling it down the hand and fold into the first glove
 3. Discard the removed gloves

4. Then, perform hand hygiene by rubbing with an alcohol-based handrub or by washing with soap and water

Important Notice

This guide is only an outline. It should not be used as the only means for selecting protective clothing. Before using any protective clothing, the wearer must read and understand the user instructions for each product. Specific country legislation must be observed. If in doubt, contact a safety professional. Selection of the most appropriate PPE will depend on the particular situation and should only be made by a competent person knowledgeable of the actual working conditions and the limitations of PPE.

Final determination as to the suitability of these products for a particular situation is the employer's responsibility. This information is subject to revision at any time. Always read and follow all User Instructions supplied with your Disposable Protective Gown in order to ensure correct operation. If you have questions contact ENMED Technical Service.

LIMITATION OF LIABILITY: Except as provided above, ENMED shall not be liable or responsible for any loss or damage, whether direct, indirect, incidental, special or consequential arising out of the sale, use or misuse of ENMED products, or the user's inability to use such products.

Central Address

Muhsin Yazıcıoğlu Cad. Rögnum
Sky Tower Kat.21 No.81
Cukurambar ANKARA- TÜRKİYE

Glove Factory

Saray Mahallesi Gıdacılar Cad. 3670 Ada 2 Parsel Kahraman Kazan /
ANKARA - TÜRKİYE